
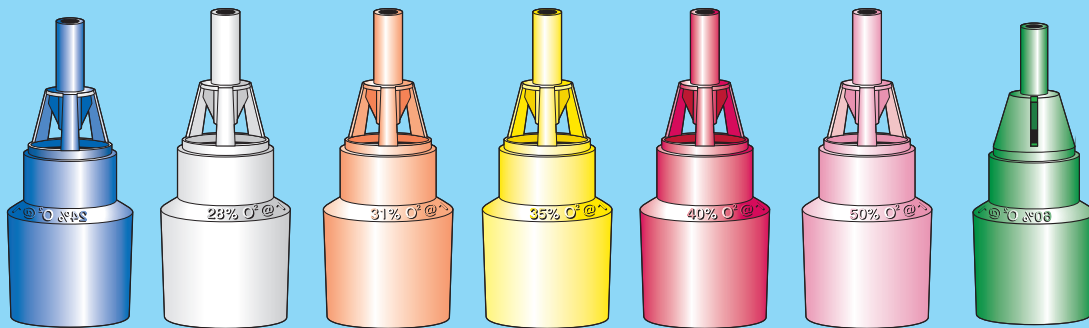


Venturi Diluter Jet Flowrate Chart

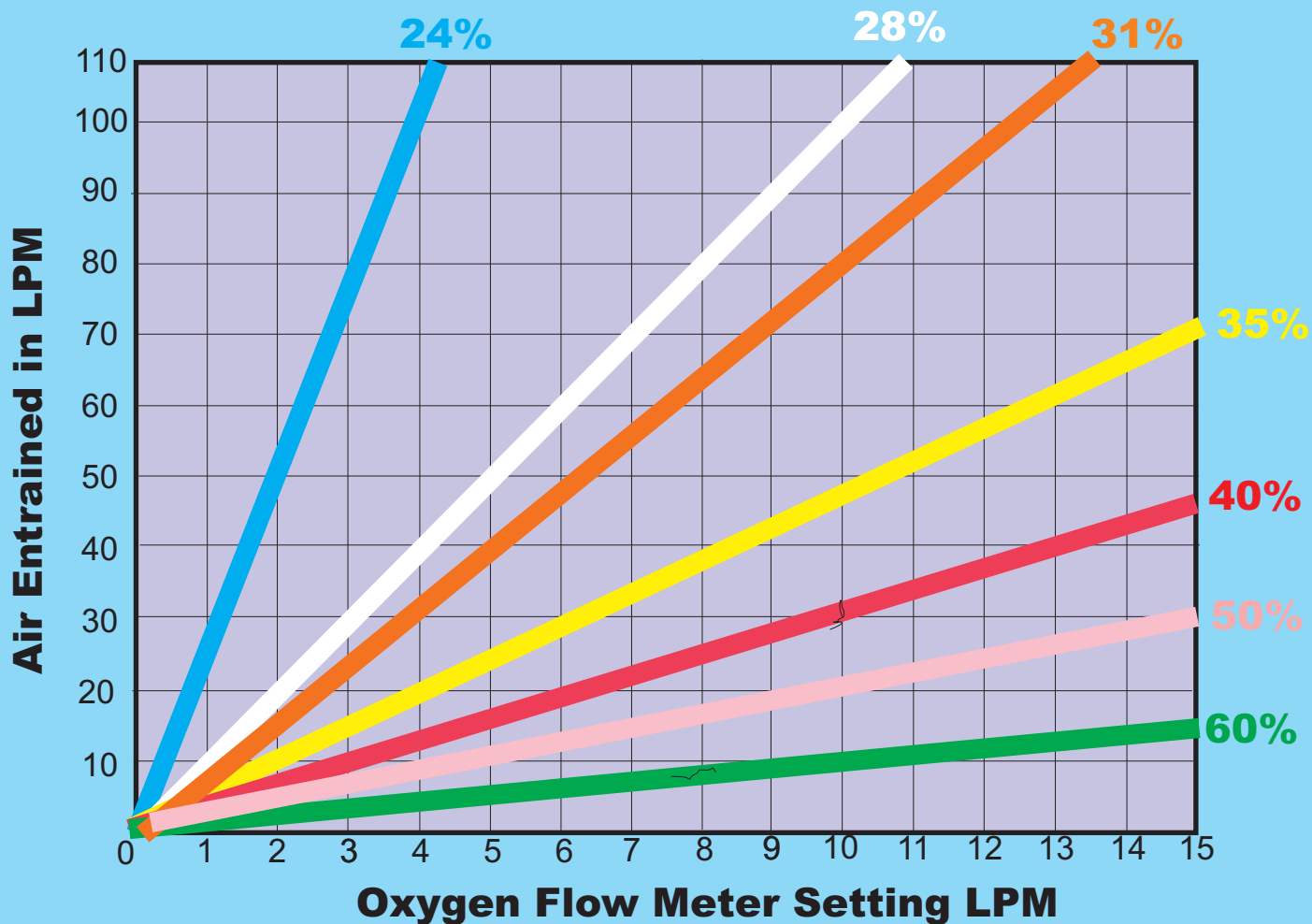

Oxygen %	24%	28%	31%	35%	40%	50%	60%
Flowmeter Set @ LPM	2	4	6	8	10	13	15
(** Range)	(1 - 4)	(2 - 6)	(4 - 8)	(6 - 10)	(8 - 12)	(11 - 15)	(14-15)
Air Entrained via Jet	50	40	46	48	30	28	15
Total Flow Delivered	52	44	52	56	40	41	30

**Venturi jets can be set to a range of flows and still deliver accurate oxygen %. Range shown above Total Flow must always be adequate for patient requirement. (Patient Inspiratory Flowrate 25 LPM average)